



Configure Communicate Page Set-up

- ✓ Click the add lab button
- ✓ You will be redirected to the 3Shape communicate website and you can add a lab by opening the connections pages and clicking add connection, search for Wiand Dental Lab.
- ✓ Wiand Dental Lab will accept the add request. (Please call if you need to expedite).
- ✓ Return to the configure communicate page and click refresh.

Direct Connect Set-up

- ✓ Click lab connections
- ✓ Click add new direct connect lab
- ✓ Enter Wiand Dental Lab and the folder path (wiandlab@gmail.com).
- ✓ Click test settings to make sure that all is correct.
- ✓ Click refresh on the labs page and Wiand Dental Lab will appear on the lab list with the suffix DC.



Initial Set-up for iTero®

- ✓ Call Align Technologies at 800-577-8767 and select option 1.
- ✓ Request that Wiand Dental Lab is added to your scanner and identify our lab using our phone number 480-446-7063 or 833-269-1195.
- ✓ After Wiand Dental Lab has been added, restart your scanner.
- ✓ After connecting to us as a lab, select Wiand Dental Lab on your scanner when sending files.



Initial Set-up for Medit

- ✓ Log in to www.meditlink.com with the administrator account.
- ✓ Go to "Partners" on the left window pane.
- ✓ Search "Wiand Dental Lab"
- ✓ Click "Request Partnership" then click "Confirm"
- ✓ You will see that Wiand Dental Lab is now listed under partners
- ✓ Wiand Dental Lab will accept the add request. (Please call if you need to expedite).



Initial Set-up for Carestream

- ✓ Visit Carestream Connect on your scanner and search for Wiand Dental Lab.
- ✓ Add Wiand Dental Lab
- ✓ Select Wiand Dental Lab when submitting files.
- ✓ Email Wiand Dental Lab at wiandlab@gmail.com.



Initial Set-up for CEREC® Primescan/Omniscam

- ✓ Create account or log in to your CEREC account.
- ✓ Select "MY CEREC CONNECT"
- ✓ Type "Wiand Dental Lab" in the box of company name and click FIND.
- ✓ Check the box to the bottom left for "Wiand Dental Lab."
- ✓ Click ADD to complete the registration.