

Digital Denture Try-In Clinical Checklist

Patient Name: _____

Date: _____

Overview

- The Try-In is an exact evaluation design of the final denture.
- The Try-In can be adjusted and equilibrated like any final denture.
- Adjustments to the Try-In should be exact instructions – expressed in millimeters of change.
- Adjustments may also be written/expressed in marker on the Try-In.

Step-By-Step Checklist

Insert Try-In in the patient's mouth and adjust sharp spots or pressure areas.

Does the Try-In have good retention? _____ Yes _____ No, new impression required

Comments: _____
Re-impression the Try-In for new scans.

Is the VDO (Vertical Dimension of Occlusion) correct? _____ Yes _____ No, needs adjustment

Comments: _____
Adjust occlusion and make new record.

Are there areas of over-extension? _____ Yes, needs adjustment _____ No

Comments: _____
Grind away any over-extension and send to laboratory to scan.

Is the midline in position? _____ Yes _____ No, needs adjustment

Comments: _____
Indicate required adjustment – left or right – number of mm.

Is the lip support adequate? _____ Yes _____ No, needs adjustment

Comments: _____
TIP: Check by looking at upper teeth on wet/dry line of lower lip. Evaluate "SH" sound for lower teeth.
Indicate adjustment in mm – facial or lingual.

Is the incisal edge position correct? _____ Yes _____ No, needs adjustment

Comments: _____
TIP: Check "F" and "V" sounds.
Evaluate whether teeth are too high or low – indicate adjustment +/- in mm.

1. Is there a cant to the horizontal plane? _____ Yes, needs adjustment _____ No

Comments: _____
Evaluate whether teeth on left or right need to go up or down in mm.

2. Is the Campers plane good? _____ Yes _____ No, needs adjustment

Comments: _____
Indicate adjustment - raise or lower in mm.

3. Is the upper tooth display good? _____ Yes _____ No

Comments: _____

4. Is the lower tooth display good? _____ Yes _____ No

Comments: _____