

Benefits of Digital Impressions

- Greater accuracy
- Permanent digital record
- Less appointments/steps
- Increased patient comfort
- Faster turnaround time
- More efficient communication

***WE CAN ALSO SCAN YOUR CONVENTIONAL IMPRESSIONS OR MODELS.
WHATEVER YOUR WORKFLOW, DIGITAL IS AN OPTION!***

Universal Scanning Tips:

- Calibrate your scanner weekly
- Turn off overhead lights
- Dry tissue before scanning
- Turn off AI feature if your scanner has it
- Use two people, one to retract and one to scan
- Move at a SLOW, steady pace as you go
- Scan everything you would need with conventional impressions
- Be sure scan is free of holes and voids, **software does not “fake out”**
- Look at scan in monochrome rather than color to see what lab sees
- Clean up your scans before sending for accuracy
- Go to vendor website for more educational resources
- Contact your scanner rep for support
- Go to YouTube for videos of techniques for your brand of scanner



FOR FULL ARCH: Be sure to verify full distal extension capturing entire vestibule back to hamular notch/retro molar pads.

*****Be sure you send an RX along with your scan*****

**Ready to
Start Scanning?**

**We can help, call Tom
today to learn more!
480-446-7063**